How to use your new Rehband Support - 7772

PL

Funkcja

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EN

Funktion

Supports and stabilises the ankle joint.

Indikation

Sprains and instability of the ankle joint.

Materiał

Neopren/SBR, plastic (PA6, HDPE/LDPE)

Applicering

- Pic.1:
Open all closures on the brace. Place it under the insole of the shoe. (If the insoles can't be taken out, see instruction X). The back of the brace shall be placed against the heel counter of the shoe.
- Pic.2:
Close the upper and lower closures. Tighten as hard as possible to still maintain a comfortable feeling.
- Pic.3:
Pull the diagonal strap through the D-ring and fasten it with the Velcro. The strap can be pulled through the D-ring either from the inside or outside the brace.

- From the outside of the foot to the inside of brace (as in picture: When the injury is on the outside of the foot. (Most common injury)
- From the foot inside to the outside of brace: When the injury is on the inside foot.

Pic.4:
Finally close the wide strap by pulling it around the ankle and tighten it so the brace feels stable. (Be careful not to tighten too hard thus stopping the blood flow)

Pic.5:
If you have a shoe where you can not remove the insole, so use the adhesive EVA pad, which is included in the package. Remove the protective film and place the pad on the plastic part that is in contact with the foot.

Place the protection directly in the shoe, with the rear against heel counter of the shoe.

Important Information

- If you have questions regarding application, please contact your local dealer or the prescriber.
- The product is not flameproof; avoid temperature above 120°C/248°F.
- Minor changes are allowed, providing the changes are made through the prescriber* or local dealer.
- The function of the product must not be changed or compromised; the product should only be used for its original, intended indication.
- Re: neoprene products, time of use should not exceed 3-4 hours. Re: products in other materials, time of use should not exceed 8 hours, unless otherwise is recommended by prescriber.
- If the product contains latex or rubber and you are allergic to latex/ rubber, we advise you to consult a prescriber before using the support. The content of the product is written above.
- If you are allergic to heat, we advise you to use a product that not contains neoprene.
- *Re: Medical- products, the prescriber must be a qualified doctor, PT, OT or Orthopaedic Engineer.

Washing Instructions

- Remove the cover from the plastic parts.
- Close all Velcro.
- Stretch product while wet and let dry.
- When the SBR/neoprene sock is dry, put it back on the plastic part.

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